

eakin Cohesive® Paste

RECOMMENDED BY 97%¹ OF FRENCH ET NURSES

At **eakin®**, we strive to help our customers live life as easily, happily and as fully as possible. We want their stoma to be just **a small part of a bigger life**.

You help us achieve this through your valued feedback, enabling us to improve and innovate. Here is a quick reminder of what nurses like you told us...

There are at least 3 **great reasons to use eakin Cohesive® Paste**

1. **Skin friendly filler** containing no alcohol in it's non-sting formulation² making it ideal for sensitive or delicate skins.



No pain or tingling when it is put on

ET Nurse: Mme Anne SIMON, Hospital Andre Mignot, Versailles, France




2. **Levels the skin surface** and helps prevent leaks by forming an effective barrier and level skin surface around the stoma, fistula or wound.

3. **Fills in crevices, skin folds and scars** helping to increase pouch wear time.



Better wear time for a complex stoma

ET Nurse: Mme Corine BONNEAU, CHU Bicêtre, Kremlin-Bicêtre, France

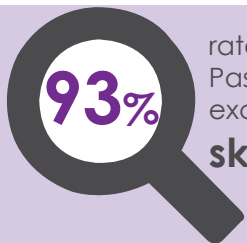

rated eakin **Cohesive®**
Paste as good or
excellent for
skin friendliness¹



of nurses surveyed will
continue to use¹



Excellent product and easy to use

ET Nurse, insert region, France



CASE STUDY

Case study submitted by:
Nadège DAUMARD/PASQUIER
IDE Stomathérapeute
Saint-Germain-en-Laye, Poissy

eakin Cohesive® Paste for pain-free skin protection

Background:

- 43-year-old lady, living in Kuwait
- January 2018: Ileostomy formed as a result of peritonitis
- April 2018: Hospitalised with severe malnutrition, an enterocutaneous fistula and abdominal wall dehiscence




Problems:

- Wound located in a hollow as well as a fistula within the wound
- Corrosive fluid leaks causing pain and discomfort
- Changes required several times per day
- Large nurse workload and additional language barrier

New care management plan:

- Key objectives: To collect effluents, heal dehiscence, reduce or eliminate pain during treatment and stop leaks and skin damage
- Application of Negative Pressure Therapy foam and adhesive film
- Alcohol-free eakin **Cohesive®** Paste applied along suture lines of fistula
- Convex ring and TPN suction cup applied
- Convex baseplate, drainable bag and belt fitted
- This procedure was done 3 times per week

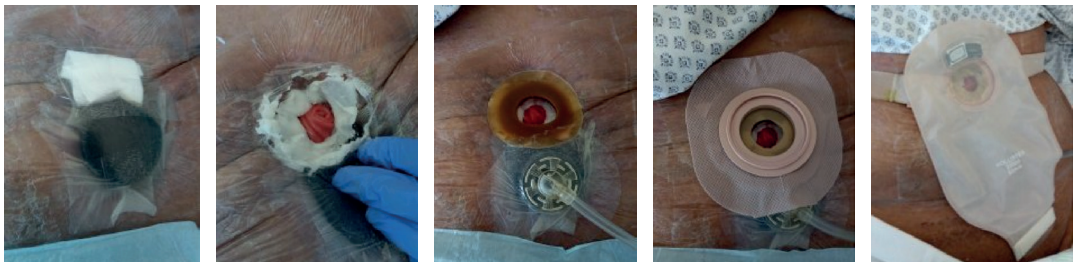

Benefits of using eakin Cohesive® Paste:

- Skin friendly filler providing effective leakage protection
- Pain-free application and removal (and no residue)

Conclusion:

- Created an optional pouching surface and more secure pouch fit. After fistula resection, lady was able to return to Kuwait.

