eakin Cohesive® paste
Medische kleefpasta

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© 2020 TG Eakin Ltd.
Rev.1 2020-01-01

EN

eakin Cohesive® paste
Medical adhesive paste

About this product: eakin **Cohesive®** paste is an alcohol-free (no-sting) adhesive paste for use in ostomy, wound & fistula care. eakin **Cohesive®** paste is intended to adhere to peristomal or periwound skin.

eakin **Cohesive®** paste helps create a level pouching surface on the skin by filling in skin folds, scars and creases around the stoma, fistula or wound, ensuring the appliance or dressing fits more securely.

eakin **Cohesive®** paste requires no drying time and is compatible with adhesives on ostomy and wound pouches allowing immediate application of the next pouch or dressing. It can be removed easily thereby decreasing user trauma or discomfort.

Instructions for use:
1. Ensure the skin in the area of application is clean and dry.
2. Separate eakin **Cohesive®** paste onto the skin, filling in any skin folds, scars or creases, and spread with your finger. Some users may find it easier to spread with a wet finger. (Apply eakin **Cohesive®** paste around the stoma or wound if applicable).
3. Place the pouch in position, ensuring a good seal with the eakin **Cohesive®** paste. (Can be used in conjunction with eakin **Cohesive®** Seals if necessary).

Remove used paste from the skin and dispose of safely in accordance with local regulations.

Warnings and cautions: Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Other information:

Store between 10 degrees and 30 degrees Celsius. Do not refrigerate.

eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to **eakin®**'s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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