## eakin protect™ barrier film spray

**About this product:** eakin protect™ barrier film is a silicone based, alcohol free liquid which can be used to protect the skin under an adhesive medical appliance. eakin protect™ barrier film is designed to prevent damage to the skin due to removal of adhesive medical appliances and to minimize skin damage from biological fluids.

eakin protect™ barrier film forms a protective layer for the skin which dries rapidly and does not inhibit the adhesion of subsequent appliances.

**Instructions for use:** Ensure the skin in the area of application is clean and dry. Spray eakin protect™ onto the skin and wait a few seconds to dry. Once dry, attach the next appliance.

**Warnings and cautions:** Pressurised container – protect from sunlight and do not expose to temperatures exceeding 50 degrees Celsius. Do not pierce or burn, even after use. Do not spray on a naked flame or any incandescent material. Keep away from sources of ignition – no smoking. Keep out of reach of children.

Avoid use on delicate areas of the body (eyes, mouth, ears, nose, genitalia). If contact occurs, flush with water and seek medical

advice. Do not inhale the spray and avoid spraying in locations where inhalation could occur. Do not use on premature infants. eakin protect™ spray is not appropriate for use in incubators.

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

**Other information:**

Store between 10 and 50 degrees Celsius. eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin®'s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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