**EN** Medical device **CZ** Zdravotnické prostředky **DE** Medizinprodukt **DK** Medicinsk udstyr **FI** Lääketieteellinen laite **FR** Dispositif médical **IS** Lækningsgæðunáour **IT** Dispositivo medico **NL** Medische toevoeging **SE** Medicinsk utrustning **SI** Medicinski pripomoček **AE** جهاز طبي

**EN** Capable of being recycled (carton only) **CZ** lze recyklovat (pouze kartony) **DE** Karton kann recycelt werden **DK** Kan genanvendes (kun kartonen) **FI** Vaidaan kierrätettävä (vain pahvi) **FR** Emballage carton recyclable **IS** Endurvinnanlegt (einselt 55%) **IT** Riciclabile (solo cartone) **NL** Geschikt voor recycling (verpakking) **SE** Kan återvinnas (endast kartong) **SI** Primerno za reciklazo (samo kartona) **AE** قابل لإعادة التدوير

**EN** Dry flat **CZ** Sušit na plocho **DE** Flach trocknen **DK** Tørres fladt **FI** Kuivaa tasoisena **FR** Sécher à plat **IS** þurrkað flatt **IT** Asciugare disteso **NL** Plat drogen, niet in de draaier. **SE** Torkas liggande **SI** Sušite poravnano **AE** لا تستخدم مجفف

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**EN** | **DE**

**About this product**  
**eakin®** support belts are lightweight fabric belts with an adjustable VELCRO® fastener. They may be suitable for those users who require hernia or abdominal support during periods of physical activity such as walking, gardening, lifting, sports, housework etc. They can be used to support a muscle weakness, especially in the first few months after surgery such as stoma surgery when the muscle fibres are trying to heal. They can also be used as a preventive measure.

**Instructions for use**  
**Choosing the correct size**  
Size selection should be ideally performed in consultation with a healthcare professional. If this is not possible please follow the steps below:

1. Lie flat on a firm surface making your abdomen as flat as possible. For those with a hernia, this allows the hernia to reduce or settle back to its normal position.
2. Using a tape measure wrapped carefully around your abdomen take a firm measurement over the area of the stoma (and hernia where applicable).
3. Note this measurement and compare it with the table below for the correct order code for your size.
4. If in doubt or between sizes, select the smaller size.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

Size	Measurement	20cm Wide Belt	26cm Wide Belt
S	75 – 90 cm	839051	839061
M	90 – 105 cm	839052	839062
L	105 – 120 cm	839053	839063
XL	120 – 135 cm	839054	839064

**Application**  
1. Apply the support belt while lying on your back on a firm surface.  
2. Start with the belt behind your back and draw the VELCRO® closures together and close them in front in a position you find comfortable.  
3. Ensure that the belt is fastened for firm but comfortable support.  
4. The support belt should be worn during periods of physical activity. It should not be worn while sleeping or lying down.

**Warnings and cautions**  
**eakin®** support belts should not be used while sleeping or lying down. Ileostomates and urostomates should not fasten the support belt too tightly, in case flow is obstructed.  
**Other information**  
**eakin®** accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to **eakin®**s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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