eakin contour™ flange extenders

Fixierstreifen | estensori curvi™ per flange | Vyrovnávací pásy | Renforts pour stomie | flansframlengingar fyrir útlínur | kantsíkring | فارتال اتيام ح تاطسرا

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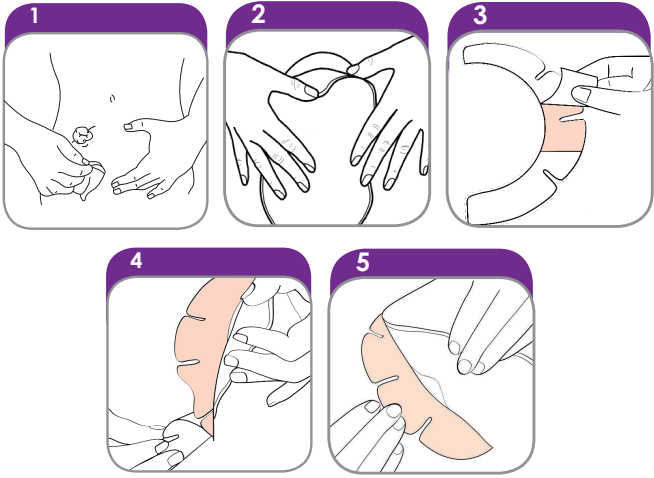
EN eakin contour™ flange extenders

About this product.

eakin contour™ flange extenders provide additional security around ostomy pouch baseplates. They may be used to help with appliance adhesion and prevent leakage problems. eakin contour™ flange extenders are compatible with any ostomy pouch or baseplate and are suitable for all body shapes.

Intended users

For ostomates, healthcare professionals and caregivers.

Instruction for use

Application

1. Ensure the skin around the stoma is clean, dry and free from any solvents, ointments or creams.
2. Apply your ostomy pouch or baseplate as instructed.
3. Remove the largest section of backing paper from the surface of the eakin contour™ flange extender.
4. Ensure that part of the eakin contour™ flange extender overlaps the edge of the baseplate. Remove the remaining section of backing paper and smooth the remainder of the eakin contour™ flange extender onto the skin to create a seal.
5. Hold in place until secure. A second eakin contour™ flange extender may be used if required.

Removal

1. Slowly peel the ostomy pouch baseplate and eakin contour™ flange extender away from the body.
2. Remove any residue by rolling and peeling. A medical adhesive remover can be used if necessary.

Disposal

Place used products into a waste bag and waste bin. Do not flush the product down the toilet. Dispose of waste products in accordance with local guidelines.

Warnings and cautions

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional.

Do not apply to broken skin or open wounds.

Do not re-use. Re-use of a single use product may have potential to harm the user due to the risk of contamination and impairment in product function.

Skin adhesion and wear time may be reduced in conditions of elevated temperature or humidity, or if the extender is re-positioned after first application.

Other information

Store between 10° and 30° Celsius. Store flat in the carton. Do not refrigerate. eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin®s current recommendations.

Any serious incident in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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© 2021 TG Eakin Ltd.