**EN** Medical device **CZ** Zdravotnické prostředky **DE** Medizinprodukt **DK** Medicinsk udstyr **EE** Meditsiiniseade **ES** Dispositivo médico **FI** Lääkinnällinen laite **FR** Dispositif médical **HR** Medicinski uređaj **IS** Lækingabúnaður **IT** Dispositivo medico **NL** Medisch hulpmiddel **NO** Medicinsk utstyr **PL** Urządzenie medyczne **RU** Медицинский прибор **SE** Medicinsk utrustning **SI** Medicinski pripomoček

securely re-joined to fit around more difficult sites.  
 4. Apply to the skin with gentle pressure and hold in place until the eakin Cohesive® seal is secure on the skin. Mould the seal towards the base of the stoma or the edge of the wound.  
 5. Ensure a snug fit, with no gaps around the base of the stoma, to provide maximum skin protection.  
 6. Place any appliance directly onto the surface of the eakin Cohesive® seal. Hold in place with gentle pressure until securely attached to the seal and the surrounding skin.

**Removal**  
 Gently remove any appliance from the body. Remove the eakin Cohesive® seal by carefully peeling and rolling from the outer edge. Wipe gently to ensure the skin is clean and dry. A medical adhesive remover can be used if necessary.

**Disposal**  
 Dispose of waste products in accordance with local guidelines.

**Warnings and cautions**  
 Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience

any of these symptoms, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Do not re-use. Re-use of a single use product may cause harm to the user due to the risk of contamination and impairment in product function.

**Other information**  
 Store flat in the carton at room temperature between 10° and 30° Celsius. Do not refrigerate.

eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin®'s current recommendations.

Any serious incident encountered in relation to this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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