


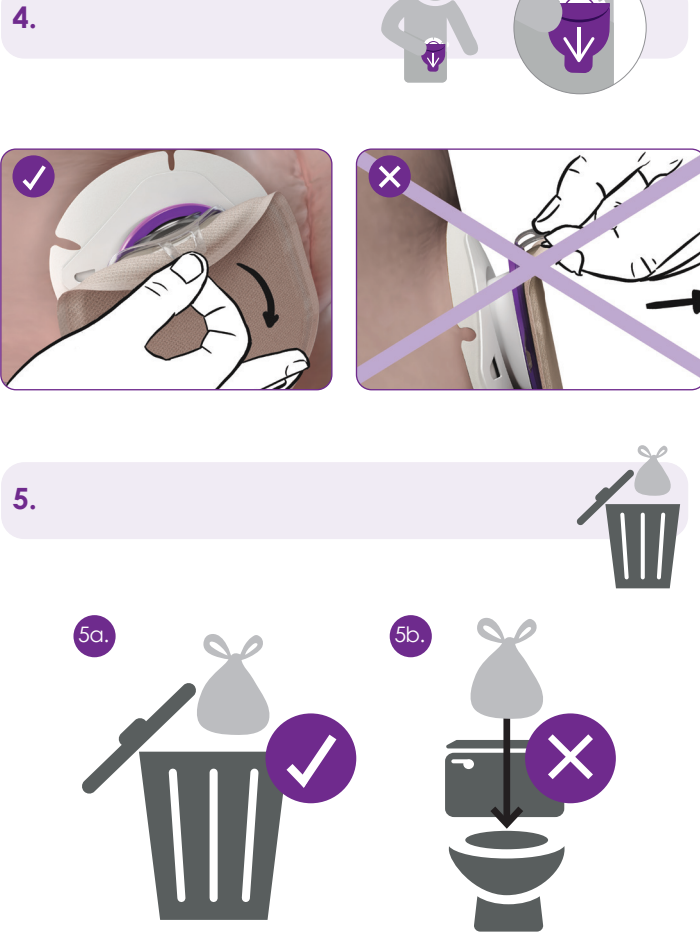


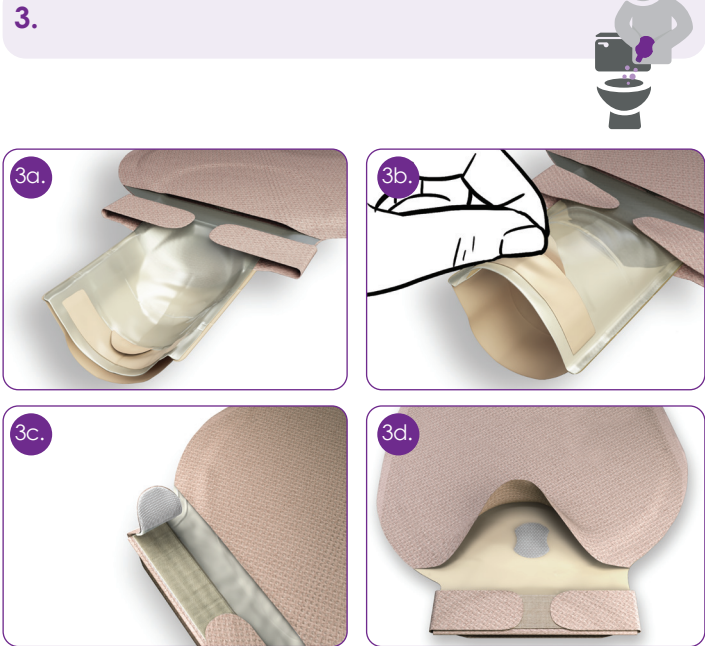

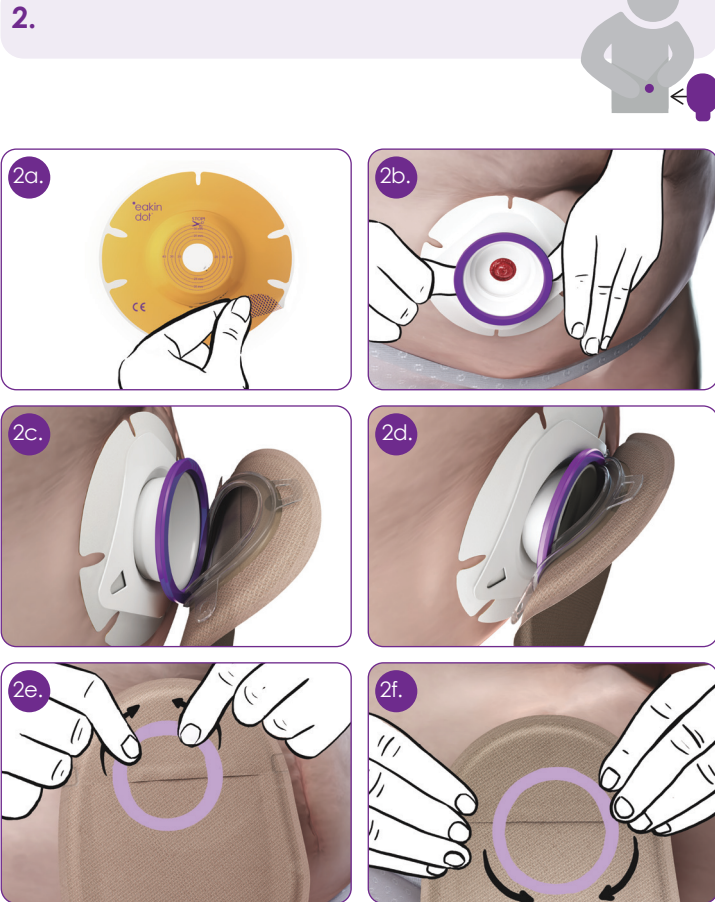

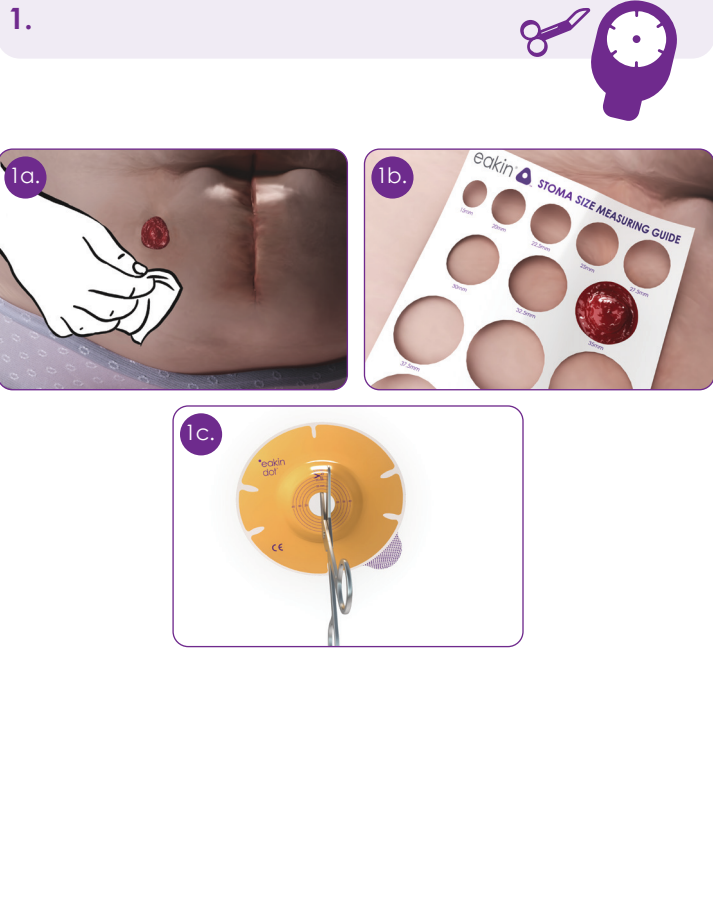

## Instructions for use

**1. Preparation:** Ensure skin is clean and dry before application (1a).

If the pouch is pre-cut, please proceed to step (2). If the pouch needs to be cut to size, use the enclosed measuring guide to measure the size and shape of the stoma (1b). Using the cutting guide on the baseplate cut a hole to fit the exact pre-measured size and shape of the stoma using curved scissors taking care not to cut through the pouch film on the other side (1c). An accurate fit reduces the risk of leaks and skin problems.

**2. Application:** Peel the protective film off the baseplate (2a). Align the centre hole of the baseplate around the stoma. Starting at the bottom of the baseplate press gently against your skin, smoothing upwards and outwards to ensure the entire baseplate is securely adhered to the skin. Extend pop-up on baseplate (2b), and click to attach pouch (2c) & (2d). Confirm secure attachment (2e) & (2f). (Filter covers are included which can be used when bathing or to prevent 'pancaking'. Apply the cover by lifting up the split fabric and placing the cover over the filter which can be seen as a dark circle).

**3. Emptying drainable pouches:** Peel both hook tabs off the Velcro strip and unfold outlet (3a). To make a larger opening for emptying pull upwards on the opening tab and drain pouch contents into the toilet. Clean any leftover output from the outlet if necessary (3b). To close, fold the outlet back up and secure by attaching the two hook tabs onto the Velcro strip (3c). Attach the Velcro strip to the stoma. Zorg ervoor dat u niet door de split fabric for added comfort and discretion (3d).

**4. (i) Removal of the pouch:** Peel down to remove (4a).

**4. (ii) Removal of the baseplate:** Gently remove the pouch by peeling the baseplate downwards and away from the skin. Apply light pressure to your skin with your free hand to aid removal.

**5. Disposal:** Place used products into a waste bag and waste bin (5a). Do not flush the product down the toilet (5b). Dispose of waste products in accordance with local guidelines.

