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eakin™

© 2023 TG Eakin Ltd.
Rev.3 Aug-2023

15 Ballystockart Road,
Co. Down, BT23 5QY,
Northern Ireland, U.K.
www.eakin.eu

CE

REF 839025

eakin Protect™ barrier film wipes

About this product: eakin Protect™ barrier film wipes contain a silicone based, alcohol free liquid which can be used to protect the skin under an adhesive medical appliance. eakin Protect™ barrier film is designed to prevent damage to the skin due to removal of adhesive medical appliances and to minimize damage from biological fluids.

Intended users: For ostomates, individuals managing fistulae or wounds, healthcare professionals and caregivers

Instructions for use: Remove the eakin Protect™ wipe from the sachet and unfold. Rub the tissue onto clean, dry skin in the area of application and wait a few seconds to dry. Once used, dispose of safely in accordance with local regulations.

Warnings and cautions: Keep away from sources of ignition – no smoking. Keep out of reach of children.

Do not re-use. Re-use of a single use product may have potential to cause harm to the user due to the risk of contamination and impairment in product function.

Avoid use on delicate areas of the body (eyes, mouth, ears, nose, genitalia). If contact occurs, flush with water and seek medical advice.

Do not use on patients less than 27 kg in weight.

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Other information: Store between 10° and 30° Celsius. Do not refrigerate. eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin®s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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