eakin Release™
limfjerningsservietter

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ADHESIVE
REMOVER WIPES

ETP/3

eakin Release™ adhesive remover wipes

About this product: eakin Release™ adhesive remover wipes contain a silicone based, alcohol free liquid which can be used to assist with the removal of medical adhesives and associated residue from skin.

eakin Release™ adhesive remover interrupts the adhesive bond to the skin, resulting in easy removal of medical adhesives and residue without inhibiting adhesion of subsequent appliances.

Intended users: For ostomates, individuals managing fistulae or wounds, healthcare professionals and caregivers.

Instructions for use: Remove the eakin Release™ wipe from the sachet and unfold. Rub the tissue over the adhesive residue to remove it. Once used, dispose of safely in accordance with local regulations.

Warnings and cautions: Keep away from sources of ignition – no smoking. Keep out of reach of children.

Avoid use on delicate areas of the body (eyes, mouth, ears, nose, genitalia). If contact occurs, flush with water and seek medical advice.

Do not use on patients less than 12 kg in weight.

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Do not re-use. Re-use of a single use product may have potential to cause harm to the user due to the risk of contamination and impairment in product function.

Other information: Store between 10° and 30° Celsius. Do not refrigerate. eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin®'s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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